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Uses: Sani-Cloth® AF Wipes are alcohol free, ready to use, low linting, disposable 2 in 1 cleaning and disinfection wipes for non-invasive medical devices; non-porous hard surfaces. Sani-Cloth® AF can be used to pre-clean or decontaminate medical devices prior to sterilisation. Latex free.

Directions: Dispense wipe and open, wipe surface in S-motion from top to bottom of area. Ensure the surface remains visibly wet for the required contact time, use an additional wipe if needed. Allow surface to air dry. Routinely remove residual disinfectant build up. For Professional Use Only.

Contra-indications: Do not use for terminal disinfection of invasive medical devices.

Caution: Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents and container in accordance with local regulations.

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Sani-Cloth® AF
CLEANING AND DISINFECTANT
LARGE WIPE

TUBERCULOCIDAL, VIRUCIDAL, YEASTICIDAL & BACTERICIDAL IN 2 MINUTES

CONTAINS: 200 Wipes
267mm x 210mm

Sani-Cloth® AF
CLEANING AND DISINFECTANT
LARGE WIPE

TUBERCULOCIDAL, VIRUCIDAL, YEASTICIDAL & BACTERICIDAL IN 2 MINUTES

ACTIVE INGREDIENTS:

Didecyltrimethylammonium chloride.....00.45%
OTHER INGREDIENTS.....99.55%
TOTAL.....100.00%

Does not include the weight of the wipe

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PDI Ltd, Pywell Road, Willowbrook East Ind Est, Corby, NN17 5KJ, UK