Label Size 300 x 145mm

Sani-Cloth®

SPORICIDAL CLEANING
AND DISINFECTANT

Chlor

LARGE WIPE

SPORICIDAL, VIRUCIDAL, BACTERICIDAL AND YEASTICIDAL IN 2 MINUTES

ACTIVE INGREDIENTS:

Sodium Hypochlorite.....	00.63%
OTHER INGREDIENTS.....	99.37%
TOTAL.....	100.00%

Does not include the weight of the wipe

Approximately 6300 ppm available sodium hypochlorite

Approximately 6000 ppm available chlorine

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Uses: Sani-Cloth® Chlor are sporicidal, high level disinfection, disposable cleaning and disinfection wipes for non-invasive medical devices; non-porous hard surfaces. These ready to use, low tinting wipes can be used to clean and disinfect in one product. Sani-Cloth® Chlor can be used to pre-clean or decontaminate medical devices prior to sterilization. Latex free.

Directions: Dispense wipe and open, wipe surface in S-motion from top to bottom of area. Ensure the surface remains visibly wet for the required contact time, use an additional wipe if needed. Allow surface to air dry. For Professional Use Only.

Contra-Indications: Do not use for terminal disinfection of invasive medical devices.

Caution: Wear protective gloves. Avoid contact with eyes, skin and clothing. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously for several minutes. If irritation occurs discontinue use. Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents and container in accordance with local regulations.

PDI Ltd, Pywell Road, Willowbrook East Ind Est,
Corby, NN17 5XJ, UK T: +44 (0) 8081 697 945
E: contact@pdi-emea.com W: pdi-intl.com

EC REP NEX Medical Antiseptics Srl
Via Per Arluno 37/39-20010 Casorezzo
(MI) Italy

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