



Frequently Asked Questions

1. Can Sani-Cloth® Detergent be used to disinfect?

No, Sani-Cloth® Detergent doesn't disinfect, it is designed for cleaning and damp dusting only. If you are required to disinfect after cleaning with Sani-Cloth® Detergent, please choose from Sani-Cloth® 70, Sani-Cloth® Chlor, Super Sani-Cloth® or Super Sani-Cloth® Plus.

2. How many Sani-Cloth® Detergent wipes are needed to clean an area, such as a mattress?

This will depend, a lightly soiled surface will require less wipes. Heavily soiled surfaces may require more wipes. As a general rule one wipe is sufficient for 1m². If the wipe becomes visibly dirty, please discard it and use an additional wipe.

3. What is the composition of Sani-Cloth® Detergent?

Description of the mixture: Contains <5% non-ionic detergent and surfactants, a light lemon fragrance and preservatives. The formulation has been designed to ensure the product is able to clean and damp dust on multiple surfaces. As the intended use for this product is to clean and not disinfect, efficacy data is not available.

4. What surfaces and equipment are compatible with Sani-Cloth® Detergent?

Sani-Cloth® Detergent contains a mild detergent for effective cleaning. The mild formulation saturated onto the wipe provides a widely compatible multi-surface wipe for cleaning and damp dusting. Follow medical device manufacturer cleaning guidelines.

5. Are Sani-Cloth® Detergent wipes a hazard to my patients and staff?

When used according to the label instructions Sani-Cloth® Detergent wipes should not present a safety hazard. The Sani-Cloth® Detergent formulation does not contain any substances that are assessed to be PBT (persistent, bio accumulative and toxic) or vPvB (very persistent and very bio accumulative).

6. What personal protective equipment (PPE) is required when using Sani-Cloth® Detergent wipes?

The use of PPE will depend on the risk assessment and the microorganisms that you may come into contact with when cleaning and disinfecting, e.g. If you are cleaning and disinfecting equipment after patient use in an isolation room, your PPE requirements may be different to those required when cleaning and disinfecting a general use ward area. PPE is required to protect Healthcare Care Professionals from coming into contact with microorganisms when cleaning and disinfecting. Please follow local Infection Control protocols at all times on the use of PPE.

7. Where can I find the expiry date for Sani-Cloth® Detergent Wipes?

Sani-Cloth® Detergent has a shelf life of 24 months from the date of manufacture. The expiration date is stamped directly on the product packaging.



8. Once open, how long do I have to use the product?

Sani-Cloth® Detergent wipes have a Period After Opening of 3 months. This is displayed on the packaging. Ensure the pack is closed when not in use, do not use any wipes which have become dry. Do not re-use any wipes.

9. How should I store Sani-Cloth® Detergent wipes?

Sani-Cloth® Detergent should be stored at room temperature away from direct sunlight.

10. How should I dispose of Sani-Cloth® Detergent wipes?

Please follow your local healthcare setting waste disposal protocols.

11. Are Sani-Cloth® Detergent wipes flushable or maceratable?

No, Sani-Cloth® Detergent wipes are not flushable nor maceratable. The DO NOT FLUSH symbol is visible on the front of the packaging.

12. Can I use Sani-Cloth® Detergent wipes if the packaging is damaged or partially open, or the lid is broken?

Sani-Cloth® Detergent wipes should not be used if the pack is damaged or partially open in any way.

13. If I require some support or training on using this product, how do I get in touch with the company?

Please refer to the contact details below.