


# Prevantics 2% CHG Applicator: Instructions For Use

Please follow local policy for personal protective equipment, hand hygiene & waste disposal guidelines.



**Step 1:** Open the packaging by pulling away the label from the corner.

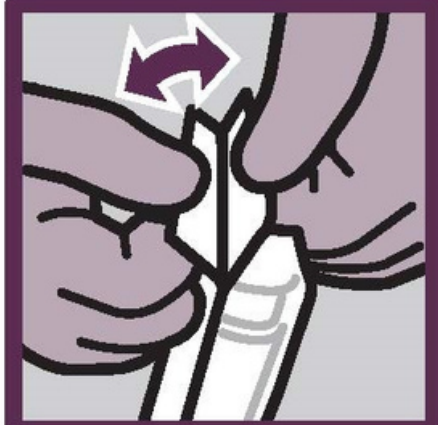

**Step 2:** Firmly press the dispenser to release the liquid from the chamber onto the sponge.



**Step 3:** Holding the applicator upright, fully press the sponge onto the skin several times to bring the liquid through the sponge.



**Step 4:** Apply the product to the area to be disinfected using vigorous back-and-forth, up-and-down, repeated strokes for at least 30 seconds. Allow the skin to air dry.

APPLICATOR SIZE	RECOMMENDED SKIN COVERAGE
26ml	50cm x 50cm
10.5ml	30cm x 30cm
3ml	15cm x 15cm



For more information, write to us: [contact@pdi-emea.com](mailto:contact@pdi-emea.com)

T: +44 (0) 8081 697 945

E: [contact@pdi-emea.com](mailto:contact@pdi-emea.com)

W: [pdi-intl.com](http://pdi-intl.com)

PDI International