

# Sani-Cloth® Detergent: Instructions for Use

## General surface cleaning and damp dusting wipes

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines

### Preparation



### Dispense



- + Dispense the wipe, pulling away from you.
- + Close the lid when not in use.

### Clean



- + If the surface is visibly soiled, clean the surface following an S-shaped motion from top to bottom.
- + Use additional wipes as needed.
- + Use a fresh wipe for each new surface.

### Dry



- + Allow surface to air dry.

### Disposal



- + Dispose of wipe after a single use.
- + Packaging is not recyclable



### Other information:

- + Store upright in a cool dry place
- + Refer to expiry date on packaging
- + Approx. 1 wipe per 1m<sup>2</sup>



For more information, write to us: [contact@pdi-emea.com](mailto:contact@pdi-emea.com)

+ 44 (0) 8081 697 945

[contact@pdi-emea.com](mailto:contact@pdi-emea.com)

[wearepdi-intl.com](http://wearepdi-intl.com)

 PDI International

