

# Prevantics® 2% Device Swab: Instructions For Use

Surface disinfectant for non-invasive medical devices.



**Step 1:** Tear open the easily accessible packaging and remove the swab.



**Step 2:** Unfold the wipe.



**Step 3:** Twist back and forth in semi-circular movements for at least 30 seconds.



**Step 4:** Ensure the connector is fully disinfected. Allow the needleless access site to air dry for 30 seconds.

# Prevantics® 2% Device Swab: Instructions For Use Device Swab



## Caution and other information:

- + Store between 20-25°C.
- + Avoid freezing and excessive heat above 40°C.
- + Refer to the expiry date on the packaging.
- + Follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines.
- + Printed version available on request.
- + Highly flammable liquid and vapour.
- + Causes serious eye irritation.



DANGER Contains Propan-2-ol

## Contra-indications:

- + Do not use for terminal disinfection of invasive medical devices.
- + Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State.

## Formulation

2% (w/w) Chlorhexidine gluconate (CHG)  
70% (v/v) Isopropyl alcohol (IPA)

For more information, write to us: [contact@pdi-emea.com](mailto:contact@pdi-emea.com)

 PDI Ltd. Pywell Road, Willowbrook East Industrial Estate, Corby NN17 5XJ UK

T: +44 (0) 8081 697 945 E: [contact@pdi-emea.com](mailto:contact@pdi-emea.com) W: [pdi-intl.com](http://pdi-intl.com)

 PDI International

