# Prevantics® 2% CHG Device

SWAB

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### ENGLISH

**Uses:** Prevantics®2% CHG Device Swabs are pre-dosed, alcohol and CHG non linting disposable disinfection wipes for non-invasive medical devices. Disinfection wipes for needleless connectors, hubs and connection ports.

**Directions:** Tear open the sachet, remove and unfold the wipe. Use the wipe to disinfect the desired surface area, ensure threads are thoroughly scrubbed. Allow surface to air dry. If surface is visibly soiled, pre-cleaning is required. Use one wipe per surface. Do not reuse wipes as this could impact the effectiveness or lead to cross-contamination. Do not use dry wipes. For Professional Use Only.

**Contra-indications:** Do not use for terminal disinfection of invasive medical devices. Not intended for disinfection of skin. Do not use for large surface areas.

**Caution:** Wear protective gloves. Causes serious eye irritation. If IN EYES: Rinse cautiously with water for several minutes. If eye irritation persists; get medical advice. Highly flammable liquid and vapour. Keep product stored in a cool dry place away from heat sources. Vapours may cause drowsiness or dizziness, if inhaled, remove person to fresh air. Dispose of contents and container in accordance with local regulations.

**For an adverse incident, please contact PDI using the details below, if applicable inform the relevant Member State where the product is sold.**

Current IFU and previous versions available on the website

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NEX Medical Antiseptics Srl. Via Per Arluno 37/39-20003 Casorezzo (MI) Italy