BLACK

PMS 7544c

475mm x 330mm

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**Uses:** Sani-Cloth® AF3 wipes are alcohol free, ready to use, low linting, disposable 2 in 1 cleaning and disinfection wipes for non-invasive medical devices and general surfaces. Sani-Cloth® AF3 can be used to pre-clean or decontaminate medical devices prior to sterilisation. Latex free. Dermatologically tested. **Directions:** Open the lid, peel away the removable seal sticker and dispose. Dispense wipe and open, wipe surface in S-motion from top to bottom of area. Ensure the surface remains visibly wet for the required contact time, use an additional wipe if needed. Allow surface to air dry. Use one wipe per surface. Do not reuse wipes as this could impact the effectiveness or lead to cross-contamination. Coverage Approx. 1m²/wipe. Do not use if wipe is dry. Routinely remove residual build up. For Professional Use Only. **Contra-indications:** Do not use for terminal disinfection of invasive medical devices. Not intended for disinfection of skin. **Caution:** Dispose of contents and container in accordance with local regulations.

UDI CODE  
45mm x 35mm

**Sani-Cloth®**  
CLEANING AND DISINFECTANT

**AF3**  
LARGE WIPE

POSITION FOR SOLO LID

TUBERCULOCIDAL, VIRUCIDAL, YEASTICIDAL & BACTERICIDAL IN 1 MINUTE

**CONTAINS: 200 Wipes**  
267mm x 210mm

**Sani-Cloth®**  
CLEANING AND DISINFECTANT

**AF3**  
LARGE WIPE

TUBERCULOCIDAL, VIRUCIDAL, YEASTICIDAL & BACTERICIDAL IN 1 MINUTE

**ACTIVE INGREDIENTS**  
N-alkyl dimethyl benzyl ammonium chloride .....0.14%  
N-alkyl dimethyl ethylbenzyl ammonium chloride.....0.14%  
**OTHER INGREDIENTS**.....99.72%  
**TOTAL**.....100.00%  
Does not include the weight of the wipe

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